

Teacher's Tips

Your FrontRow Pro Digital has an energy-saving STANDBY mode.

The receiver will automatically go into STANDBY mode 10 minutes after you turn off or mute your microphone or stop playing a secondary audio source (Computer, TV/DVD, CD, etc.)

The receiver will come out of STANDBY mode automatically when:

- You turn **on** and/or **un-mute** a microphone
- You play an audio source connected to the receiver

START OF DAY

Step 1 Put microphone around neck and adjust strap so it is 5in (13cm) from your mouth.

Step 2 Turn microphone **on**. This will also take the receiver out of STANDBY mode

Step 3 Speak in a normal conversational tone. Adjust receiver volume to a comfortable level.

END OF DAY

Step 1 Turn microphone **off**. **Note:** Receiver will go into STANDBY mode automatically. You do not need to turn it off.

Step 2 Charge overnight by plugging it into the charge cord on the back of the receiver.

BATTERY TIP

Batteries cause the majority of troubleshooting problems. If your system doesn't appear to be working, try replacing the batteries with fresh cells first.

LIGHT LEGEND

Microphone




Green Power On
Blinking Green Muted
Blinking Red Battery Low

When Charging:

Off Alkaline Battery Detected

Red Charging
Blinking Red Maintenance Charge

Receiver

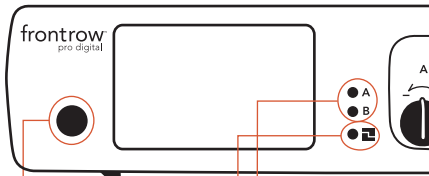

Green Power On
Yellow Standby

Green Signal Received
Red No Signal/Muted

Green Feedback suppression active